PL- Inhalator kompresorowy -tłokowy
GB- Compressor piston nebulizer
DE- Inhalator-Kompressor
RU- Поршневой компрессорный ингалятор

OMNIBUS

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Shenzhen Bi-rich Medical Devices Co.Ltd,
 No. 10th, Hongqian 2nd Road ,ShangNan, Shajing street, Bao'An district,
 518125 Shenzhen City, Guangdong Province,P.R.China

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Utilization8

IEC symbols

Warning. Act with caution. Read the instruction ...	
Type of equipment	
Alternative current	
Insulation class 2	
For indoor use.	

Precautions

Warning:

1. The medical equipment is for use only after consultation with a doctor or a physiotherapist.
2. Follow the instruction manual and the instruction of your physician or doctor to operate unit.
3. Inhalations should be recommended by the doctor who is able to prescribe the suitable medicine, its dose and the way of administering.
4. The equipment is intended for the aerosoltherapy only. Another usage than pointed out in this instruction is inadvisable.
5. Do not untwist the container for medicine (a nebuliser) during the inhalation.
6. Do not pour more than 6ml of solution into the container (nebuliser jet).

Danger! Electrical protection

1. Always unplug the device from power after the inhalation.
2. Do not use the nebulizer under the shower, in the bath, etc.
3. Use and storage the nebulizer far from baths, sinks, washbasins and other moisture sources.
4. Do not immerse in water and other liquids. Do not switch on the wet nebulizer!
5. If some water get into the apparatus – immediately unplug the power and do not use!

Warning: Minimizing the risk of the electric shock, fire and other damage.

1. The risk of electric shock! Do not remove the cabinet or open the cover.
2. Always disconnect the power cord from the electrical outlet before cleaning or servicing.
3. Do not storage and use the nebulizer within easy reach of gas, flammable, explosive materials or open fire.
4. Do not use any greases and oils.
5. Always switch off the apparatus after finishing the inhalation.
6. Storage the power cord far from the warmed, hot places.
7. Do not throw, do not put any other elements into the holes of the nebulizer.
8. Do not cover, clog air intake grilles on the enclosure.
9. Do not use in the wet or moisture rooms.
10. Unplug the power before filling the container with a medicine.
11. Using the nebulizer near TV sets, microwave ovens, mobile phones or other devices which emit strong magnetic field. It can disturb their working.

Warning: Preventing the infections

6. The equipment predicted for administering medicines by the way of nebulisation. Due to its small size and friendly appearance it is recommended for children and adult patients.
7. It is recommended to clean the accessories after each aerosol treatment. Conduct the disinfection one a day. Please follow cleaning and disinfecting instructions in this manual.

The nebulizer is not suitable for use in anesthetic respiratory systems.

The nebulizer is not predicted for giving thick suspension or oily substances. Before the aerosol therapy please know the instruction of the medicine administering.

Introduction

The nebulizer is designed to deliver a prescribed medicine in prophylactic and therapeutic aims for treatment of the respiratory tracts such as asthma, allergies, pulmonary, colds and others.

The nebuliser converts the medicine solution into an aerosolized mist, which is inhaled through the mask or mouthpiece. The apparatus is a modern and very efficient device for using at home.

Read the instruction manual before using.

Specification

Power	230V AC/50Hz
Power consumption	below 60W
Rated current	0.7 A
Dimensions	150 x 140x 90 mm H
Weight	1.2 kg
Noise level	50 dBa
Maximum compressor pressure	2.41 – 3.45 bar (241-345 kPa)
Operating pressure	0.62 – 1.10 bar (62 – 110 kPa)
Maximum flow	7 – 11 l/min.
Liter flow range	5 – 8 l/min.
Partical size less than	3 microns
Maximum nebulizer solution capacity	6 ml
Operation conditions	temperature: +10°C to + 40°C humidity : 30% do 85% RH
Storage conditions	temperature: - 20°C to + 70°C humidity: 10% do 95% RH

Designed for continuous operation.

PA protection against an electric shock.


 Isolation class II

BF class of enclosed accessories: a mouthpiece, a mask

The safety level at presence of flammable anesthetics or oxygen – the lack of AP/APG (do not use at presence of flammable anaesthetics or oxygen)

IPX0 – protection against harmful in gress of water is Ordinary

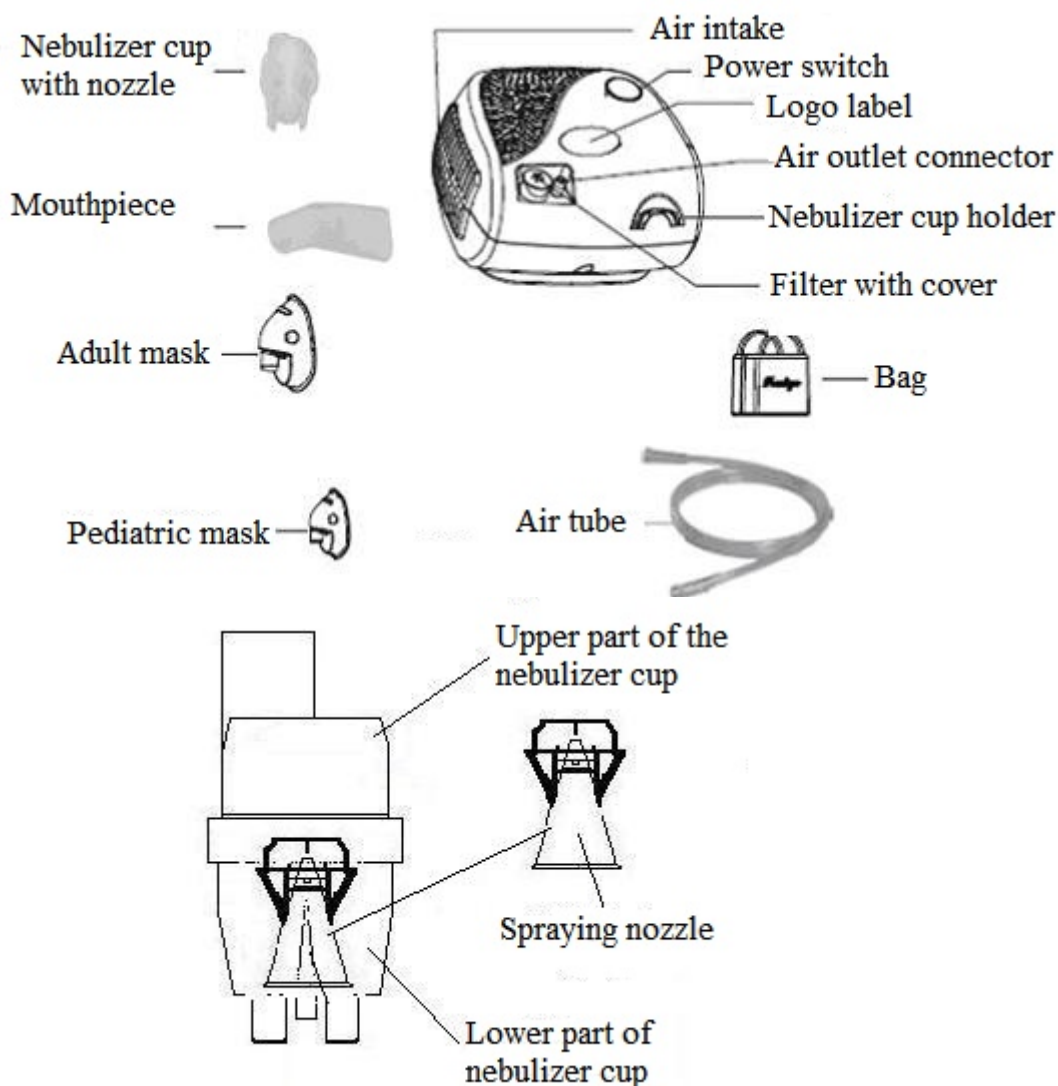Description

WARNING: There should be close attention that without the spraying nozzle, the nebulizer DOES NOT WORK !

ACCESSORIES:

The compressor

The mask for adults, the children's mask, the air tube, the mouthpiece, the nasal tip

The medicine container (nebulizer cup) with the spraying nozzle, the complete of 5 filters, the bag

Operation instruction

1. Take out nebulizer from the bag and put on the stable, flat, hard surface.
 2. Take out the power cord, make sure that the device is switched off (position O of the power switch).
 3. Plug the end of the power cord to the nebulizer, put the plug into the socket.
 4. Connect one end of the air tube into the air outlet connector on a nebulizer's body.
 5. Unscrew the nebulizer cup (medicine container), check if the SPRAYING NOZZLE is inside. Pour the measure dose of medicine, prescribed by the doctor.
- The sign 6 cc on the container points out the maximum container's capacity. Do not pour more than maximum level of medication solution. Tighten the container.

**WITHOUT THE THE SPRAYING NOZZLE the NEBULIZER WILL NOT SPRAY A MEDICINE!
DO NOT LOSE THE COVER PLATE OF THE NOZZLE.**

6. Put the mouthpiece or the mask on the medicine's container (nebulizer cup).
7. Put the second end of the air tube on the bottom part of the nebulizer cup, from the bottom. Sit down.
8. Switch on the device (position I of the power switch).
9. Put the mouthpiece between teeth, clench lips around the mouthpiece and breath deeply with a mouth.
10. When you use the mask, close attention to suit it to your face. Breath slowly and deeply, doing short breaks after each inspiration and doing fast expiration.

Inhalations should be conducted on the pitting position!

After finishing operations, the device should be switched off and taken out the plug from the socket.

Cleaning

1. Switch off the device and take out the plug from the socket.
2. Disconnect the air tube from the outlet.
3. **CLEANING:** Disassemble the mouthpiece and the nebulizer cup together with the spraying nozzle. Clean all elements in hot water with addition of the mild detergent. Rinse them thoroughly and let air dry.
4. **DISINFECTION:** Prepare the solution: 1 part of white vinegar and 3 parts of hot water. Immerse the mouthpiece, mask, spraying nozzle and the medicine container in the solution for 0.5 h. Remove the solution let air dry.
5. There is no need to clean the air tube. Rub the inhaler's body with a soft, dry cloth.

WARNING: The nebulizer cup (medicine container) should be changed if it got dirty or plugged.

WARNING: Do not boil, do not scald out the medicine container and the other accessories.

Maintenance

1. The air filter in the compressor should be changed if it only starts to darken. In order to do that raise up the filter's cover, take out the used filter, put inside the new filter and shut the cover. Use only original filters and accessories! Additional filters are available at the Seller or on the website: www.sklep-omnibus.pl
2. Repairs can be made only by the authorised service. The service in Poland: Omnibus Medical Equipment, 9 Grunwaldzka St., 82-500 Kwidzyn, www.omnibus-med.com.pl

WARNING: The nebulizer can be damaged, if it is used with the dirty or unoriginal filter.

DO NOT USE WITHOUT THE AIR FILTER.

POSSIBLE PROBLEMS AND REMEDIAL MEASURES

Problem	Reason	Solution
After switching on- the device does not work	Does the conduct plug to the socket?	Check and plug the power cable to the socket
The lack or the weak nebulisation during the device's operation	Is the medicine in the container?	Pour the suitable amount of the medicine to the container.
	Too much or too little of medicine in the container?	Minimum 2 ml maximum 6 ml
	Lost, unmounted the spraying nozzle in the medicine container?	Mount the spraying nozzle inside the medicine container.
	Is the set of accessories compounded correctly?	Check and mount the accessories in accordance with the instruction manual.
	Is the air outlet hole in the lower part of the medicine container plugged?	Check and clean the outlet hole (nozzle) in the lower part of the medicine container.
	Is the medicine container with the mask or the mouthpiece tilted at the acute angle?	Make sure, that accessories set (nebulizer cup + mask or mouthpiece) is not tilted at the angle more than 45 degrees
	Is the air tube connected right to the compressor?	Plug air tube properly to the compressor and to the lower part of nebulizer cup.
	Is the air tube folded or damaged?	Make sure, that the air tube is correct, not folded. If necessary- replace the tube.
	Is the air tube plugged?	Make sure that there are no contaminations in the air tube.
	Is the air filter dirty?	Replace dirty filter for a new one.
The nebulizer works unnaturally loudly?	Does the filter's cover on the compressor's mounted correctly?	Mount correctly the air filter's cover plate on the compressor.
The nebulizer hits up very much	Are air vents holes (air intake) in the compressor body covered?	During operation- don't cover air vents (air intake) in the compressor body.

If none of the solutions brought the result, do not try to repair the device on your own! The repair must be made by the manufacturer or the authorised service.

Service contact: tel.+48 55 2615092– before sending the equipment to repair, phone and consult with the technician!

WARRANTY

Omnibus Medical Equipment provides 3- year warranty from the date of the nebulizer's purchase. In this time the change of elements which submit guaranty repairing and labour are free. Only original spare parts are covered by the warranty. All modernizations, modifications without the agreement of the manufacturer and the importer are forbidden under penalty of loss of the warranty!

The manufacturer and the importer are not responsible for damages which are the result of the improper use of the apparatus and using unoriginal accessories.

The warranty does not cover the normal consumption of the equipment during usage and individual plastic accessories such as masks, a fastener, a mouthpiece, an air tube, a medicine container, filters.

The product should be delivered on the own cost together with the warranty card, the proof of purchase and the damage description directly to the firm Omnibus-Serwis, 9, Grunwaldzka St., 82-500 Kwidzyn, Poland.

All costs connected with the safe packaging, security, transport and another risk are incurred by the user.

The nebulizer is predicted for the home use and therefore the warranty does not cover the apparatus used in a professional way in health centres, clinics, hospitals, etc.

The warranty time is prolonged in case of the necessity of import the necessary parts on the base of the written information sent to the Customer by the Service.

In the range of matter which is not regulated by the conditions of herein warranty, the provisions of the Civil Code are applied.

Technical data

Particles' size: approx. 3 μm (MMAD)

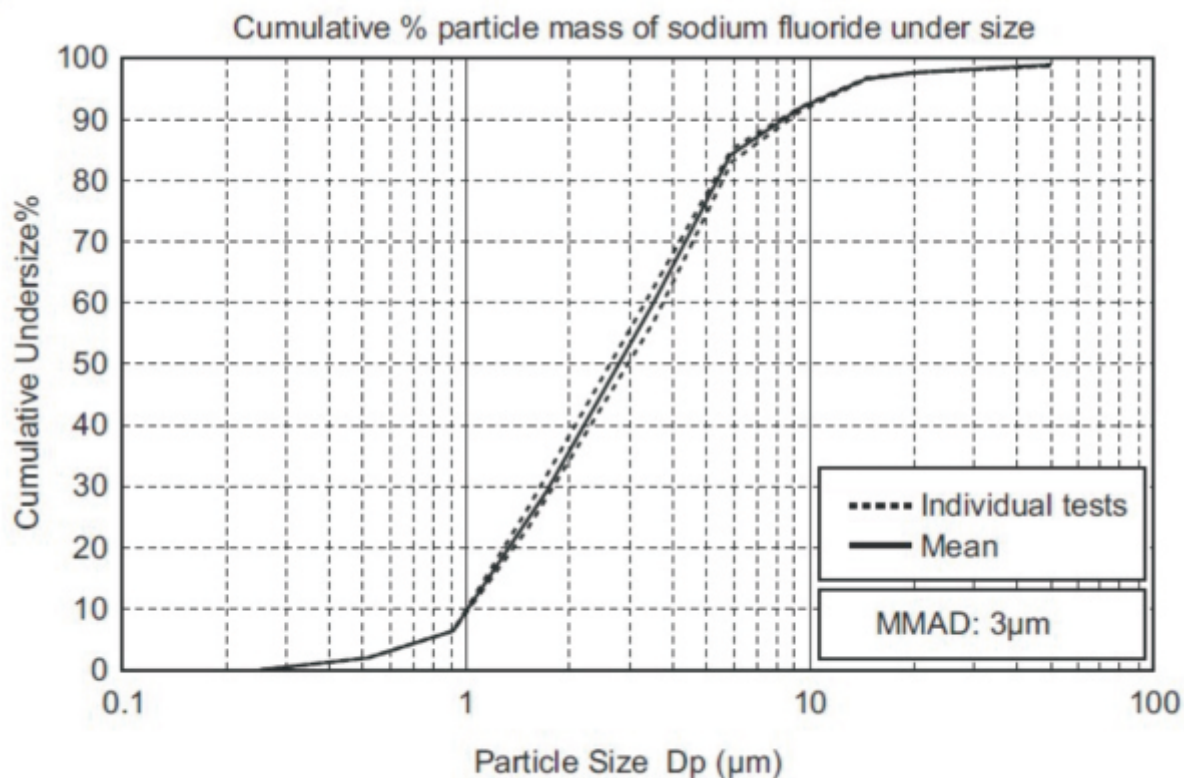
The medicine container's capacity: min.2 ml max. 6 ml

Nebulisation rate: 0.35 ml/min.

Aerosol output: over 0.33 ml (2 ml, 1% NaF)

Aerosol output rate: 0.06 ml/min. (2ml,1%NaF)

Result of cascade impactor – measurements for particles size with Omnibus CN-116 nebulizer and nebulizer kit Omnineb.

MMAD= Mass Median Aerodynamic Diameter

Disposal

The used equipment should be utilized in accordance with the European Directive 2002/96/EC. The device should be delivered to the special container or the specialized waste utilization point. Marking with the emblem of the crossed – waste bin informs about the necessity separate storage.

Manufactured for:

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A.Kozłowska-Laskowska i A.Laskowski s.c.

ul.Grunwaldzka 9, 82-500 Kwidzyn

Poland

www.inhalatory.com.pl

tel.55 2615092

Shenzhen Bi-rich Medical Devices Co.Ltd,
No. 10th, Hongqian 2nd Road ,ShangNan, Shajing street, Bao'An district,
518125 Shenzhen City, Guangdong Province,P.R.China

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Shenzhen Bi-rich Medical Devices Co., Ltd,